


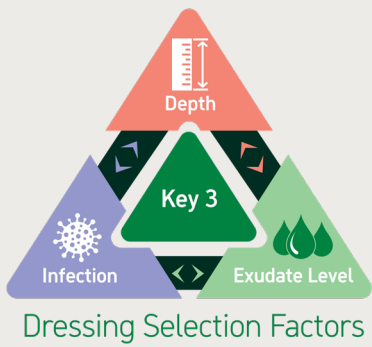

Wound Management Dressing Selection Pathway

Cleanse all wounds and debride if appropriate

Step 1

If wound has depth,* consider use of Exufiber®/Exufiber® Ag+ as a primary dressing

*and is of moderate-to-heavy exudate

Step 2

Infected or high risk of infection?

• Redness

• Swelling

• Warmth

• Odor

• Purulent Drainage

• Increased Exudate

• Pain or Tenderness

No

Yes

Select dressing based on exudate level

Exudate Level | Moderate to High

Exufiber®

+

Mepilex® Border Flex

Exufiber®

+

Mepilex® Up

Exufiber®

+

Mextra® Superabsorbent

Mepilex® Transfer

+

Mextra® Superabsorbent

or

Mepilex® Up

+

Mextra® Superabsorbent

Mepitel®/Mepitel® One

+

Mextra® Superabsorbent

Mepilex® Border Flex

Mepilex® Up

Exudate Level | Moderate

Mepilex® Border Flex

Mepilex® Up

Mepilex®

Mepilex® Border Flex Lite

Exudate Level | Low

Mepilex® Border Flex Lite

Mepilex® Lite

Exudate Level | Moderate to High

Exufiber® Ag+

+

Mepilex® Border Flex

Exufiber® Ag+

+

Mepilex® Up

Exufiber® Ag+

+

Mextra® Superabsorbent

or

Mepilex® Transfer Ag

+

Mextra® Superabsorbent

Mepitel® Ag

+

Mextra® Superabsorbent

Mepilex® Border Ag

Exudate Level | Moderate

Mepilex® Border Ag

Mepilex® Ag

Exufiber® Ag+

+

Mepilex® Border Flex

Exufiber® Ag+

+

Mepilex® Up

Exudate Level | Low

Mepilex® Ag

In case of dry wound with stable eschar, consult a wound clinician

Need to protect fragile skin?

Mepilex® Border Flex Lite

Mepitel®/Mepitel® One

Mepilex® Transfer

This guide was created under the direction of this customer and is only for the use of this customer. The information provided herein is not to be construed as the practice of medicine or substituted for the independent medical judgment of a patient's treating clinician. This information, including but not limited to suggestions for product selection and suggested use is based on generalizations and does not consider the unique characteristics of an individual's condition and/or wound. Each patient's clinician shall remain solely responsible for assessing the severity of patient's condition and/or wounds, determining the appropriate management of the condition and/or wound. For additional information, please refer to the applicable product insert.

1-800-843-8497 | www.molnlycke.us | 5445 Triangle Pkwy, Ste 400, Peachtree Corners, GA 30092

The Mölnlycke, Mepilex®, Exufiber®, Mepitel®, and Mextra® trademarks, names and logo types are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Distributed by Mölnlycke Health Care US, LLC, Peachtree Corners, Georgia 30092. © 2025 Mölnlycke Health Care AB. All rights reserved. 1-800-843-8497.

