


Skin tear algorithm of care

Mölnlycke® has developed a simple classification guide to skin tears in accordance with the proposed standardisation of identifying and documenting skin tears produced by ISTAP.

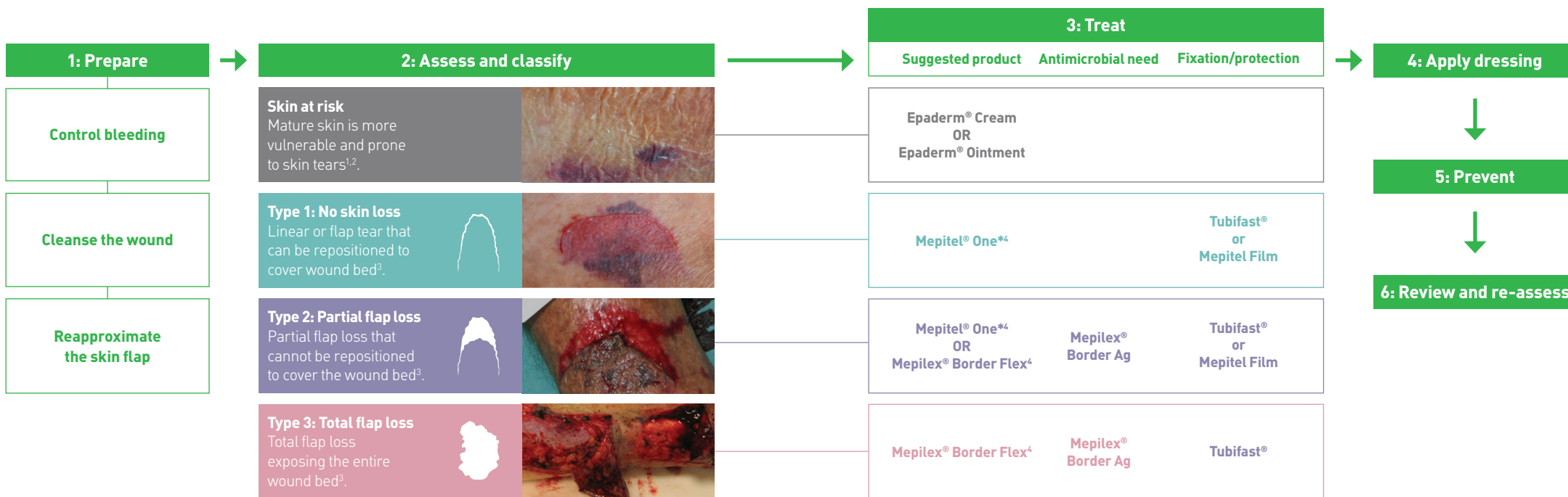

*Plus secondary dressing Mesoft® or Mesorb®.

References: 1. White MW. et al. Skin tears in frail elders: A practical approach to prevention. Geriatric Nursing, 1994. 2. Resnick B. Wound care for the elderly. Geriatric Nursing, 1993. 3. LeBlanc K. et al. International skin tear advisory panel: A tool kit to aid in the prevention, assessment, and treatment of skin tears using a simplified classification system. Advances in Skin and Wound Care, 2013. 4. Woo K. & LeBlanc K. A randomised controlled pragmatic study to evaluate the use of silicone dressings for the treatment of skin tears. Poster presentation at Wound Con Summer (virtual) conference, 2020.



Scan the QR code to read more about skin tears management or visit [L.ead.me/bbWN04](https://www.lead.me/bbWN04).

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, Gamlestadsvägen 3C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00. The Mölnlycke, Mepitel, Tubifast, Mepilex, Epaderm, Mesoft and Mesorb trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. ©2020 Mölnlycke Health Care. All rights reserved. HQIM002128

