

Mepilex® Border Flex Application Guide

Prepare



Start by cleansing the wound in accordance with normal procedures and make sure the surrounding skin is **completely dry** before application

NOTE

Select an appropriate dressing size according to the wound size by making sure that the wound pad will overlap the wound by at least 1-2 cm.



Open the sterile packaging and remove the dressing



Do not stretch the dressing while applying and avoid wrinkles

Apply

3" x 3" and 4" x 4"



1. Remove the outer release film (purple bar)



2. Apply dressing to wound and smooth border



3. Repeat on other side and smooth dressing into place

6" x 6" and 6" x 8"



1. Remove the central part of the release film (two purple bars)



2. Apply dressing to the wound



3. Release each side film and smooth dressing into place

Maintain

0 Sides



Strikethrough at 0 edges.
Can keep in place

1 Side



Strikethrough at 1 edge.
Can keep in place

2 Sides



Strikethrough at 2 edges.
Can keep in place

3 Sides



The dressing is saturated when the strikethrough has reached 3 edges.

NOTE

- The dressing may be left in place for up to 7 days
- Inspect the dressing and surrounding skin daily
- Leave the dressing on unless there are clinical reasons to change
- Change the dressing
 - If exudate leaks from the dressing or into the borders
 - If there is any sign of infection
 - When the dressing is fully saturated (exudate reaches 3 out of 4 sides)

Mepilex® Border Flex

Frequency of change

Mepilex Border Flex may be left in place up to 7 days, depending on the condition of the wound and surrounding skin, or as indicated by accepted clinical practice.

Areas of use

Mepilex Border Flex is designed for a wide range of acute and chronic wounds such as leg and foot ulcers, pressure ulcers, traumatic wounds and skin tears. It can also be used on dry/necrotic wounds in combination with gels.

Precautions

In the case of signs of clinical infection, consult a healthcare professional for adequate infection treatment. Do not use on patients with known sensitivity to the dressing or its components.

Mepilex Border Flex ordering information†

Product Code	Size	Pcs/box	Pcs/case	HCPCS
595200	3" x 3" (7.5 x 7.5 cm)	5	50	A6212
595300	4" x 4" (10 x 10 cm)	5	50	A6212
595400	6" x 6" (15 x 15 cm)	5	50	A6213
595600	6" x 8" (15 x 20 cm)	5	50	A6213

† Packaged sterile in single pack



Go to www.connect2know.com for cutting-edge info on wound management, latest evidence and FREE Nursing Continuing Education courses to support best practice across your organization, and webinars presented by experts in hospital acquired condition prevention, health economics, pressure injury etiology and other essential topics.

Find out more at www.molnlycke.us/MBFlex

The Mölnlycke Health Care, Mepilex and Safetac trademarks, names and logo types are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2018 Mölnlycke Health Care AB. All rights reserved. 1-800-843-8497. MHC-2018-T00006

270600 11.18

